LAMBOO

MOBILE MEDICAL

**SITE PLANNING
GUIDE**

**Covid-19 CT
Emergency scan box
Siemens Somatom GO**

PROJECT NUMBER.: 209047

Table of Contents

1.	General	1
2.	Site Requirements.....	1
3.	Support Pad Requirements	2
4.	Electrical Requirements	3
5.	Radiation Shielding	4
6.	Telephone & Computer connection	4
7.	Water connection	5
8.	Airflow considerations	5

Annex: Drawing Exterior views 209047-003
Drawing Site requirements 209047-005
Drawing Support points 209047-006 rev-A
Drawing Radiation protection 209047-007
Drawing Measurements weight 209047-850 rev-A

#	REVISION	DATE	NAME
-	Draft version	15-04-2020	D. Hordijk
A	First release	16-04-2020	D. Hordijk
B	Support points modified	22-06-2020	D. Hordijk

Important Notice.

The information in this document is subject to change without notice.
Lamboo Specials Sales B.V. assumes no responsibility for any errors or omission appearing in this document. Contact Lamboo Specials Sales B.V. for the latest planning information.

Lamboo Specials Sales BV.
Koperstraat 44
2718 RE Zoetermeer.
The Netherlands.

Tel.nr. +31 79 363 83 83
Fax.nr. +31 79 363 83 10
E-mail : info@mobile-medical.eu
Website : www.mobile-medical.eu

1. General

This medical trailer requires specific site preparations by the operator at each desired location.

This guide contains information about the necessary preparations of the site, before placement and connections of the medical trailer.

The site must meet these requirements before the mobile unit can function properly.

2. Site Requirements

The minimum area required for parking the trailer (without the tractor cab) is

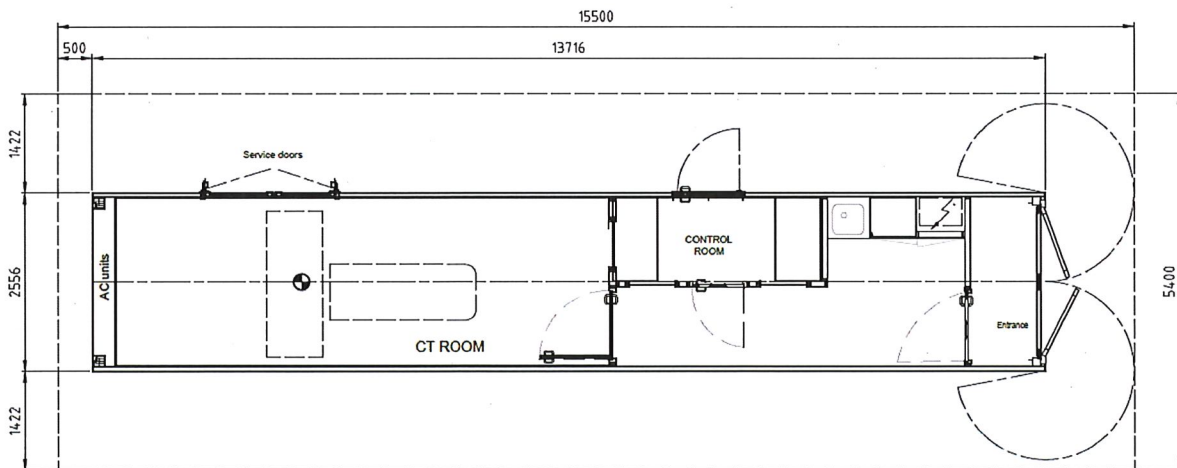
Length= 15,5 m

Width= 5,6 m

The ground and the access way must be clearly paved.

In order to ensure proper operation of the stabilizing system the ground must be level and the deviation must not exceed 1% (which is 0,6 degree!).


Make sure that that the entrance and service doors are easily accessible for patient entry and services.

Drawing 209047-005 gives an overview of the dimensions and weight of the Mobile Medical unit. (see appendix for full drawing).

3. Support Pad Requirements

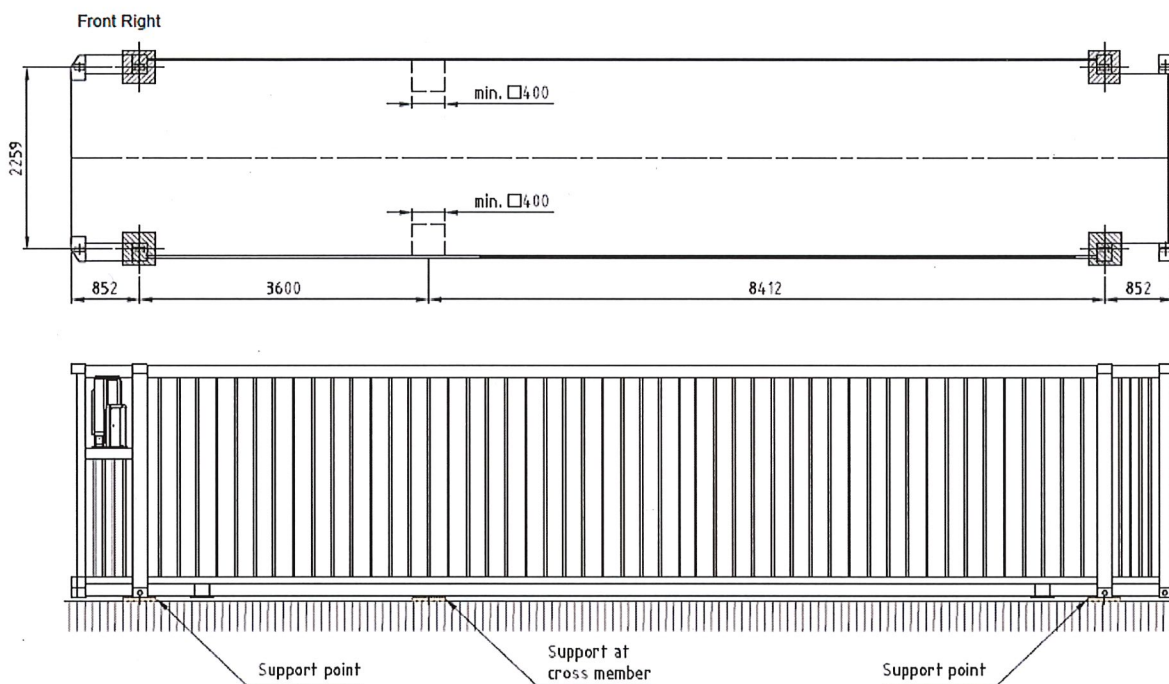The ground and access way must be able to carry the weight of the trailer and tractor.
 The weight of the Scan box WITHOUT scanner is approx 14 000 kg.
 The weight of the Scan box with CT scanner is approx 16 000 kg.
 Under the corner castings and in the middle must be placed a nylon support block of 0,4 x 0,4 m.
 Drawing 209047-006 gives the positions of support points.

IMPORTANT NOTICE:

In order to ensure proper operation of the unit the support points must be as level to the zero as possible. This can be done by using shim plates of aluminium or nylon.
 Good practice is to operate all doors. If the doors are stuck then the unit is NOT level.
 Re shim the unit until all doors can be opened without a problem.

Container floor has step of approx. 240 mm to enter the unit.
 Lamboo does not provide any ramp for access with patient trolley.

4. Electrical Requirements

The trailer is equipped with a 400 volt 3 phases 50 Hz electric system according to the TN-S system.

The site shall provide a 400 V, 50 Hz, service fused at the following capacities:

		Fuse rate:
1.	Somatom GO CT	160 A

The electrical phase rotation of the supply should be L1 L2 L3.

A 15 meter shoreline cable, wired directly to the trailer, is provided for connecting the mobile unit to local power.

The mains lead is not intended to withstand the wear and tear of traffic passing over it. The site shall provide necessary protective ramps, catenary systems or other means to protect the cable where it traverses roads or pathways.

The shoreline cable is provided with a with a 380 V - 250 A Marachell DS2 male connector with 2 parts:

- 1) Marachel part # 39 28 017: Inlet (male)
- 2) Marachal part # 392A 013: Handle

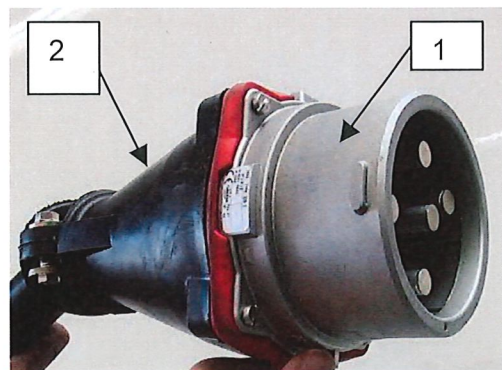
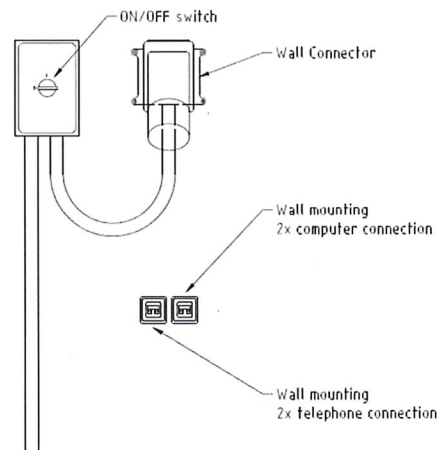
The customer facility must have the female connector to match this connector.

The electric installation must provide a dedicated medical earth terminal for connecting of the earth connection of the trailer.

This earth cable shall be connected to the trailer and the main supply outlet before the mains cable is plugged in!

WARNING:

It's the user's responsibility to verify that the shore power is electrically compatible with the connector of the mobile trailer. Connecting the power to the wrong type of receptacle could cause serious injury or damage of the medical system.

5. Radiation Shielding

The trailer is equipped with a lead protection structure. Despite of this measure, low level of radiation can exist outside of the mobile unit. The trailer should be placed in an area that keeps the exposure minimal to outdoor personal and to the environment.

See radiation shielding drawing for values.

6. Telephone & Computer connection

The unit is equipped with a patch panel RJ45 type for telephone computer patching.

If necessary the unit can be connected to the hospital IT network.

The site must supply cables for connecting to the IT network.

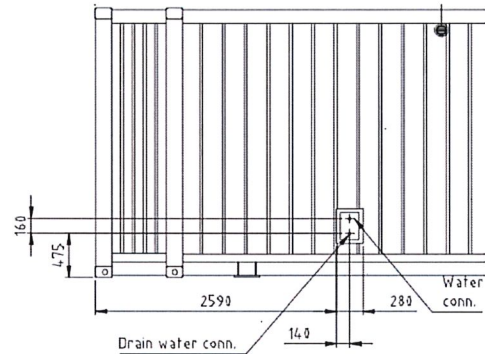
7. Water connection

The unit is equipped with a direct connection for fresh water and for waste water connection. The location of the connection points is at the outside with connections to the shore with a hose.


Connector for shore line water is GK for hose size 19 x 3 mm.


8. Airflow considerations

The following clearances are acceptable for the airflow: 1000 mm from the grills of the outside airco unit and generator.

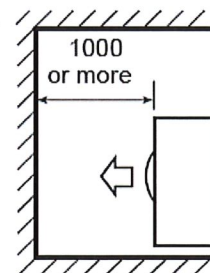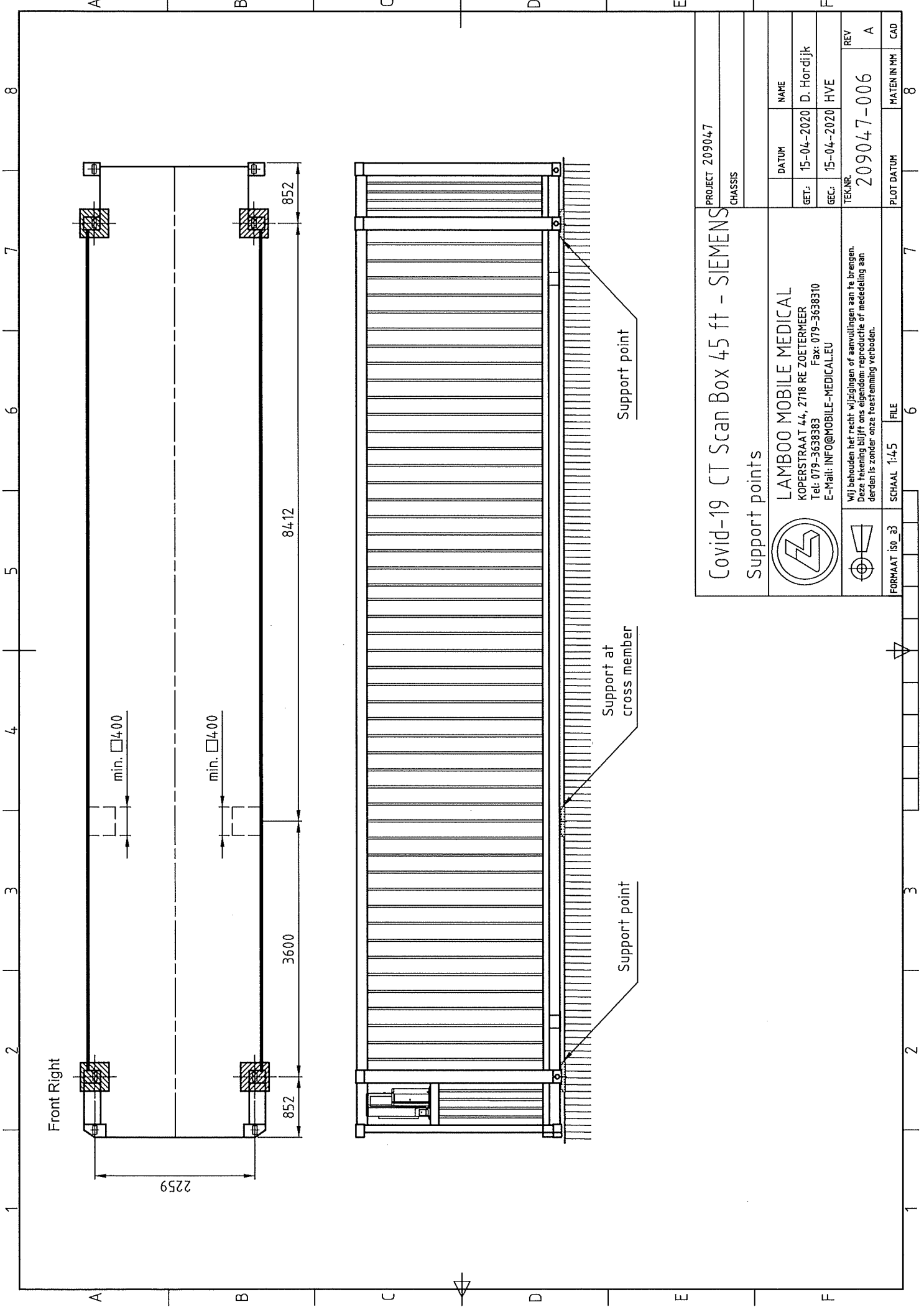At the front side the condenser system is situated. The grills for the condenser need the following clearance for the airflow:
-1000 mm for the front grills


PROJECT 209047	
CHASSIS	
DATUM	NAME
GET.: 15-04-2020	D. Hordijk
GE.C.: 15-04-2020	HVE
TEK.NR.	REV
209047-006	A
PLOT DATUM	MATEN IN MM
	8

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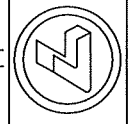
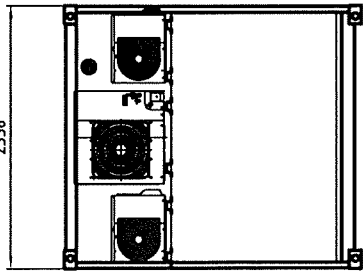
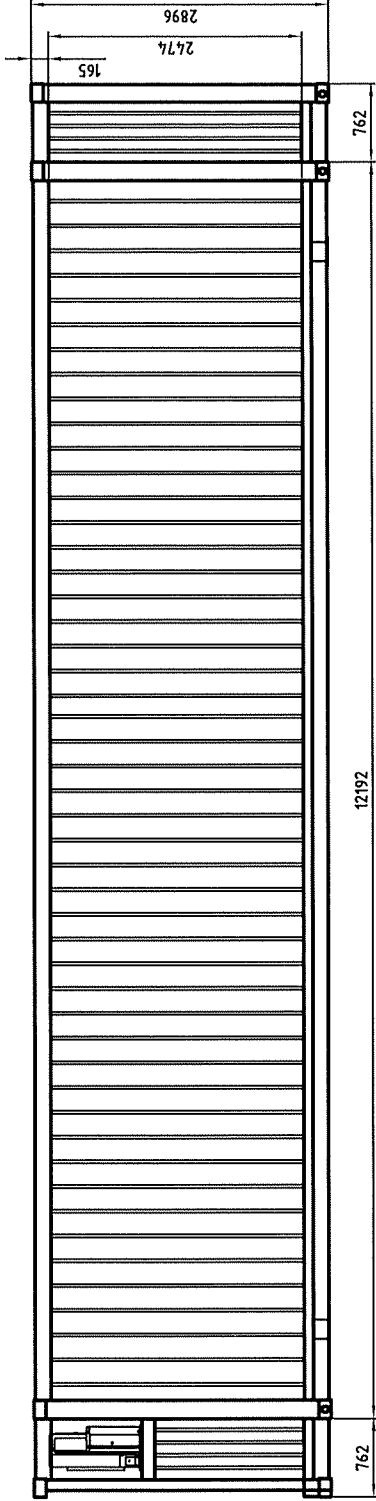
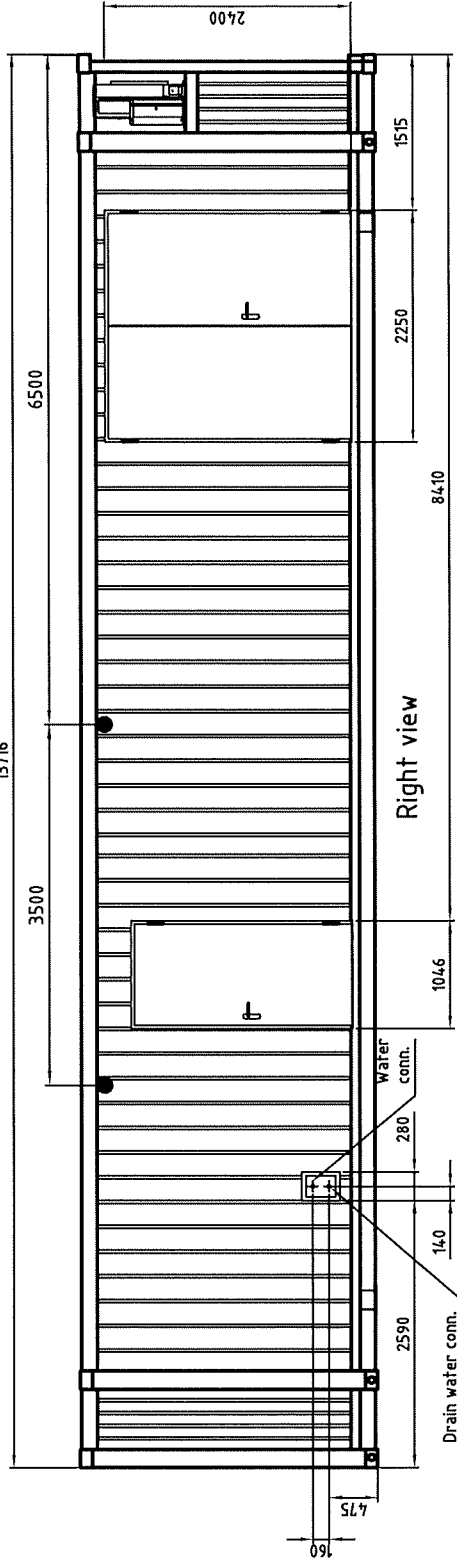
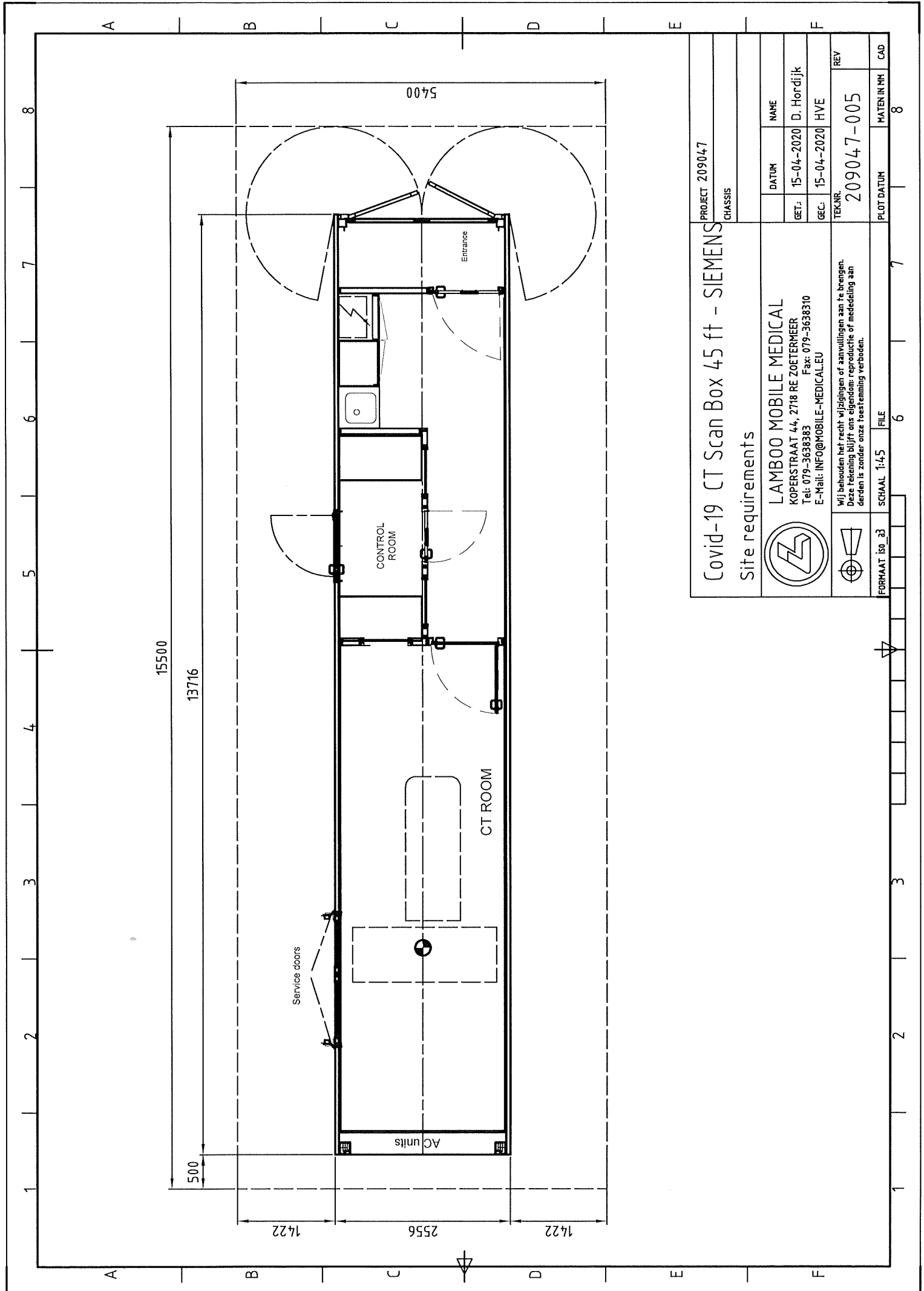
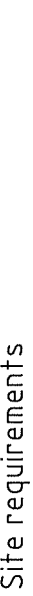
PLOT DATUM MATEN IN MM CAD

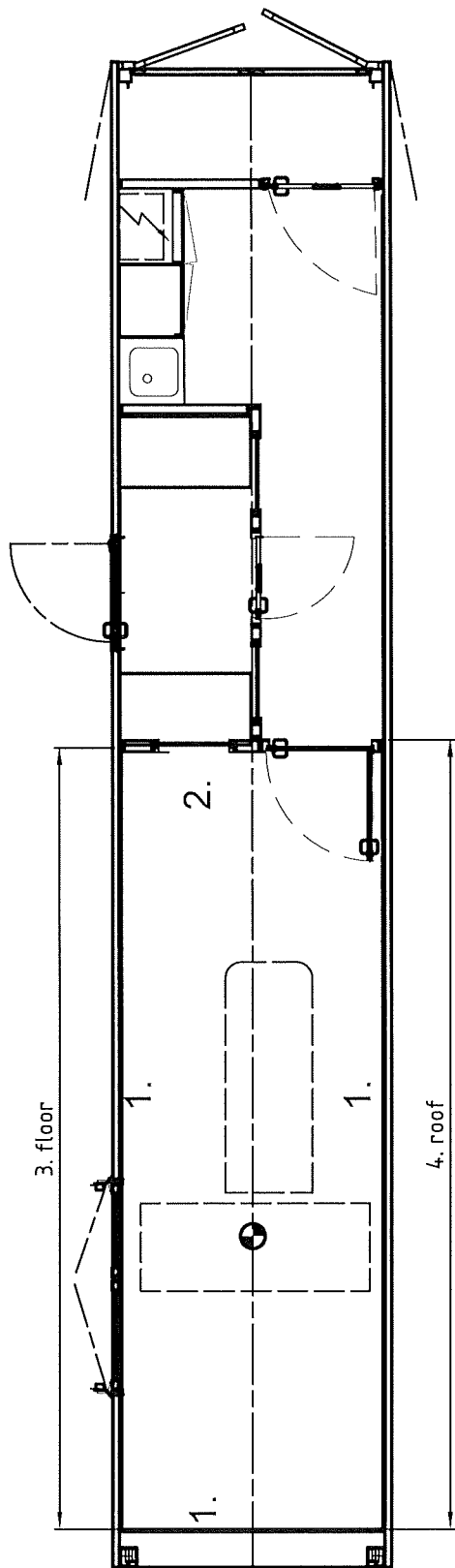8

- 1. EXTERIOR WALLS EXAMINATION ROOM: 2.5 mm LEAD
- 2. SEPERATION WALL CONTROL ROOM: 2.5 mm LEAD
- WINDOWS SEPARATION WALL CONTROL ROOM & DOOR: 20 mm Steel
- 3. FLOOR EXAMINATION ROOM: 1.32 mm LEAD
- 4. ROOF EXAMINATION ROOM:

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